


Endoform™

PATIENT GUIDE

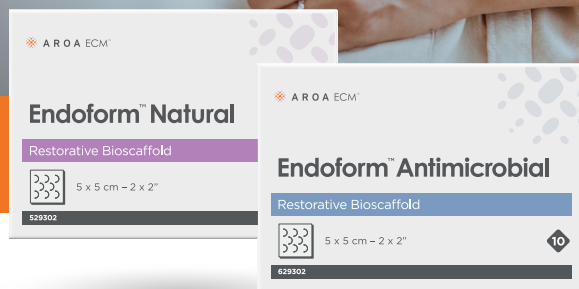

Wounds are often painful, can be hard to heal and can affect our quality of life. Your health care professional has decided that your wound needs some extra help healing.

Endoform™ is designed to help support your body during the different phases of wound healing. A bioscaffold made from ovine forestomach, **Endoform** helps your tissue in the affected area to begin to stabilize and rebuild healthier tissue so you can heal.

This guide has been provided by your health care professional with their instructions for how to care for your wound.

How to apply

Apply _____
pieces of **Endoform**
every _____ days.

Always use as instructed by your health care professional and according to the Endoform instructions for use.

If you have any questions or concerns, contact them immediately.



Hints & Tips

- Wash your hands before and after you change your dressing. Use gloves where possible
- When you change your dressing, if you notice the **Endoform** has dried out, please add sterile water or saline to hydrate it
- Keep the wound clean, and protect it from anything that can bump it



When to contact your health care professional

- If your wound bleeds or leaks more than normal
- If your wound gets red, hot or swollen, or if it starts to smell
- The wound is becoming more painful
- If you have any other concerns regarding the use of the product or your wound



Instructions For Use

Natural

Antimicrobial



More Information

www.aroa.com

1-877-627-6224



Rinse and cleanse wound bed as instructed by your health care professional.



Remove **Endoform** from package, to cut to the shape of wound using clean scissors, and place on wound.



Hydrate **Endoform** with saline or sterile water, for up to 60 seconds.



Apply a secondary dressing as directed by your health care professional.

Clinical Instructions:

NOTE: Specific indications, contraindications, warnings, precautions and safety information exists for **Endoform**. Please consult a doctor or nurse and product **Instructions For Use** prior to application. This document may be provided to patients/caregivers at the discretion of their healthcare professional and is not intended to replace the **Instructions for Use**. Intended for use within the U.S. only.

AROATM, Endoform™ Natural, Endoform™ Antimicrobial and AROA ECM™ are trademarks of Aroa Biosurgery Limited.



ARO A™



ARO A BIOSURGERY LTD

MKT:1887.00 | © March 2023