

# Myriad Matrix™

## Recommended Burn Application

### Deep Partial Thickness Burn



#### SURGICAL DEBRIDEMENT AND PLACEMENT

- Thorough debridement and wound bed preparation
- Trim 2-layer Myriad Matrix according to size of wound
- Hydrate Myriad Matrix in saline
- Fix in place using standard techniques



#### SECONDARY DRESSING

- Non-adherent dressing, stapled on corners
- Hydrogel
- +/- Softsorb®
- Kerlex™ roll
- Ace™ wrap



#### FIRST 5-7 DAYS

- Keep non-adherent dressing in place
- Re-apply hydrogel daily
- Change Kerlex™ +/- Softsorb® and Ace™ wrap, daily



#### ON DAY 5-7

- Take down everything, including non-adherent dressing



#### UNTIL CLOSURE

- Daily dressing change
- Bacitracin
- Non-adherent dressing



#### CLOSURE

#### Additional Considerations:

**RINSE WOUND  
BED WITH  
HYPOCHLOROUS  
ACID**

**DAY 4, 7-8  
NON-ADHERENT  
DRESSING FULL  
TAKE DOWN**